



# Sharp Units

## - Correct Waste Segregation



### Yellow Lidded Sharps Unit

For sharps contaminated with medicinal products (**NOT cytotoxic or cytostatic**).



### Orange Lidded Sharps Unit

For sharps **not** contaminated with medicinal products, for example phlebotomy sharps and blood sample vials.



### Purple Lidded Sharps Unit

For **cytotoxic and cytostatic** sharps, e.g. chemotherapy drugs, Depo-Provera, Chloramphenicol.



NO FREE LIQUIDS

**Do not** use for the disposal of free liquids.



Sharps units should be filled to **maximum** of three - quarters full **OR** to the fill-line marked on the container, whichever is reached first.