

# Bio Systems Container User Guide

## 30.2 litre (foot pedal) containers

Bio Systems reusable containers arrive assembled and ready for use. Containers should be installed in a secure area.

Bio Systems reusable containers have different closing mechanisms to single use containers – please read the following instructions carefully.

### To open

1. Slide the red locking mechanism to the rear of the container.  
Engage the trolley's opening device by hooking the lip around the red lock of the container.



### Waste disposal

2. Firmly press on the foot pedal to open the lid.  
Drop waste vertically into the container.  
Do not fill past the maximum fill line on the container.



### Temporary closure

4. To activate temporary closure press lid firmly at both sides and push the red lock to engage the locking mechanism.  
To reopen, slide the red locking mechanism to the rear of the container.



### Final closure

5. When  $\frac{3}{4}$  full, place thumbs close to both sides of the red lock and press down firmly. Then push the frame forward and lift out the container.



**NEVER** place Bio Systems containers in wheeled bins.