

Bio Systems Container User Guide

Horizontal lid (or 'tip-tray') containers

Bio Systems containers are reusable and have different closing mechanisms to single use containers – please read the following instructions carefully.

Waste disposal

1. Place the waste onto the tray.
2. Close the lid – this will cause the tray to tip the waste into the container.
3. Do not fill over the maximum fill line, as indicated on the container.

Temporary closure

4. To temporarily close the container ensure the tray is tipped into the container and the lid is down, then engage the two red clips on either side of the lid.

To unlock, open the two clips on either side of the lid and allow the lid to rise, making the tray visible.

Final closure

5. When $\frac{3}{4}$ full, to ensure the container is finally closed carry out step 3 above and then slide the large red tab on the front of the bin to the LOCKED position.

Correctly carrying the container

6. Always carry the container by using the handle located near the red tab.

NEVER place Bio Systems containers in wheeled bins.

Bio Systems containers arrive assembled and ready for use. Containers should be installed in a secure area, e.g. a lockable wall bracket or other suitable location.

