

# Sharps disposal guide

Is the sharp  
cytotoxic or  
cystostatic?

NO

Is the sharp contaminated  
with medicines?

YES

Dispose of in a  
purple-lidded Bio  
Systems  
container

YES

Dispose of in a  
yellow-lidded Bio  
Systems  
container

NO

Dispose of in an  
orange-lidded  
Bio Systems  
container



NO FREE LIQUIDS

**Do not** use Bio Systems containers for the disposal of free liquids.



Bio Systems containers should be filled to maximum of three-quarters full OR to the fill-line marked on the container, whichever is reached first.