

Bio Systems Container User Guide

Vertical drop (or 'funnel-neck') containers

Bio Systems reusable containers arrive assembled and ready for use. Containers should be installed in a secure area, e.g. a lockable wall bracket or other suitable location.

Bio Systems reusable containers have different closing mechanisms to single use containers - please read the following instructions carefully.

WASTE DISPOSAL

1. Drop waste into the container.
2. Do not fill over the maximum fill line, as indicated on the container.

TEMPORARY CLOSURE

3. To temporarily close the container, ensure the black sliding mechanism is in the 'open' position (i.e. slid as far anti-clockwise as possible)
4. Close the red lid, push in the front red lip until it clicks into place.

FINAL CLOSURE

5. When $\frac{3}{4}$ full, to ensure the container is finally closed carry out steps 3 and 4 above and then slide the black sliding mechanism clockwise to the 'lock' position.

The edge of the black sliding mechanism on the right hand side of the lid will fit **OVER** the red cap to prevent it from being opened. Do not force the cap.

Please note, the mechanism isn't locked unless black is visible in the small gap in the lid.

CORRECTLY CARRYING THE CONTAINER

6. Always carry the container by using the handle located between the red tabs.



These containers **MUST NOT BE PLACED IN WHEELIE BINS.**



We protect what matters.

© Stericycle Inc., 2023